



BABYBELLE


BREAST SHELLS

Model Number: BBP11



Product Introduction & Manual

1. BREAST SHELLS PARTS DESCRIPTIONS

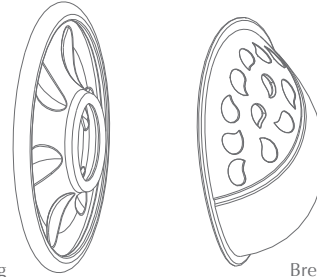

Silicone Backing

Breast Shell


2. NOTE

- Please read this manual carefully before using the product and keep it properly.
- This product can be cleaned by water.
- Do not expose the product to high temperatures, such as sunlight or heaters.
- Please stop using the shell if any discomfort occurs during the usage.
- **Please consult a health care consultant or breast-feeding consultant for correct usage of this product. To reduce the discomfort caused by incorrect usage.**

3. HOW TO USE

- Prior to the first use of this product, please read the manual carefully and keep it for future reference.
- This collector is designed to prevent the overflow of breast milk. It is a safe and sanitary product which enable mothers to feed more easily.

Product assembly as shown below :

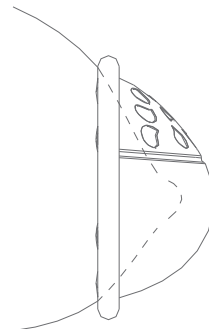

- Clean the Shell before use and keep it clean and dry.
- Wear the shells for half an hour before breast-feeding to allow the gentle pressure exert around the areola and induce slow flowing of milk. This would help to relieve the engorgement for better and easier breast-feeding.
- Assemble the silicone backing to the domes. Please make sure the domes are covered by the silicone backing totally to prevent any leakage.
- Slide the shells in to the bra with the cone surrounded your nipple in the center and fit the backing shape to your breast tightly and comfortably. Ensure the air circulation holes are on top after you fixed the shells to provide good air circulation around the nipple at all times.
- Please pay special attention that the collector and silicon should fit the breast tightly to prevent overflow of breast milk.
- If any leakage happens, please insert a breast pad at the lower edge of the shell dome.

4. HOW TO CLEAN

- Before first use, disassemble every part and place them in a bottle sterilizer or boil them for 10 minutes.
- After every usage, please clean all the parts with mild soap and clean with warm water. Rinse and dry them completely.

5. PRODUCT FEATURES

- Help to treat the soreness with the air circulation designed.
- Help to release the engorgement during pregnancy or breast-feeding period.
- Reduce the discomfort cause by the chafing between the nipples with clothes.
- Easy to use and clean.

6. WHO SHOULD USE

Mothers who are in pregnancy or breast-feeding period. Mothers who have engorgement problems or sore nipples problems.

SAVE THESE INSTRUCTIONS

www.bb-babybelle.com