BABYBELLE

smartly care

Please Save This Instruction Manual

Manufacturer : Tai Yu International Manufactory Limited
Address : South Seas Centre Tower II, 8/Floor,
Unit 804 & 805A 75 Mody Road, TST East,
Kowloon, Hong Kong

Warranty period :

You can help protect the environment! Please remember to respect the local regulations: hand in the non-working electrical equipments to an appropriate waste disposal center.


Breast Pump Instruction Manual

Model : BBP10
Input : DC6V 0.8A
Power : 2.4W

Breast Pump Instruction Manual

Model : BBP10

1. Parts Description

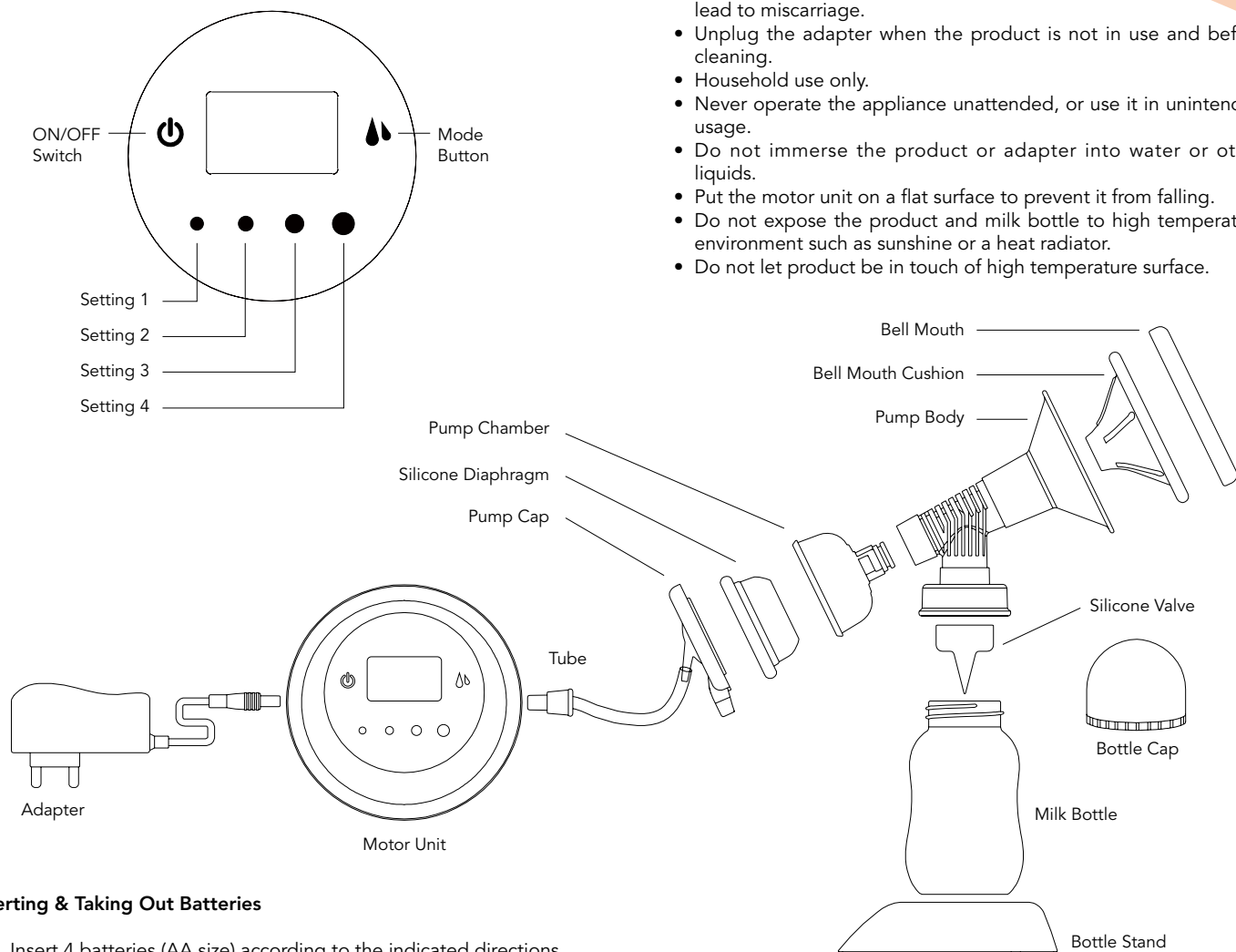Inserting & Taking Out Batteries

1. Insert 4 batteries (AA size) according to the indicated directions.
2. Pull the tape to take out the batteries.

2. Safety Precautions

- Read this instruction manual carefully before using the product, and save it for future reference.
- Before plugging in the product adapter, check that the electrical rating shown on the rating plate corresponds with your household electricity supply.
- Only use the adapter provided with the product.
- This appliance is not intended for use by persons (including children) with reduced physical sensory or mental capabilities, or lack of experience and knowledge, unless they have given supervision or instruction concerning use of the appliance by a person responsible for their safety.
- Do not use appliance during the period of pregnancy as it may lead to miscarriage.
- Unplug the adapter when the product is not in use and before cleaning.
- Household use only.
- Never operate the appliance unattended, or use it in unintended usage.
- Do not immerse the product or adapter into water or other liquids.
- Put the motor unit on a flat surface to prevent it from falling.
- Do not expose the product and milk bottle to high temperature environment such as sunshine or a heat radiator.
- Do not let product be in touch of high temperature surface.

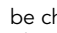
3. How to Use

Before the using the breast pump for the first time, please spend some time to go through this user manual, and save it for future reference.

This is designed for expressing milk in a safe and convenient way. The amount of expressed milk depends on the physical condition of the mother.

Expressing Mode		
Setting	Time (min)	Milk Vol. (ml)
1	10	82
2	10	100
3	10	120
4	10	150

Breast Pump:

1. Clean and prepare properly before expressing milk.
2. Press and hold the ON/OFF button for 2s, the display flashes 3 times and shows the working time of the last operation. Then it will start working with the mode and speed of the previous operation (the memory feature only works when the power is not interrupted between operations).
3. During the operation, the mode can be swapped between Massage Mode  and Express Mode , and the speed can be changed from Setting 1 to Setting 4.
4. The speed setting should be adjusted until the user feels comfortable.
5. Special Note: The Bell Mouth Cushion should be tightly pressed against the breast to prevent the milk from leaking or poor suction performance.
6. Special Note: Do not directly pull out the breast pump from the breast when the product is operating in Express Mode. Switch off the product and use a finger to pry open the seal between the Bell Mouth Cushion and the breast.
7. Using for the first time, press and hold the ON/OFF button for 2s. The display flashes "00:00" 3 times, and the product starts working in Massage Mode and Setting 4. If no button is pressed within 2 minutes, the product will change to operate in Express Mode and Setting 2.
8. The product is powered by AA x 4 1.5V batteries, or the Adapter. If the Adapter is plugged in with the batteries inserted in the product, the default power source is the Adapter.
9. Special Note : Confirm the input voltage and the output voltage of the Adapter.
10. Please protect the environment. Do not expose batteries to sunshine. Never dispose batteries in fire or country side. Avoid using old and new batteries together. Follow the local legislation for environment protection. Dispose of the product and batteries at an appropriate waste treatment centre.

4. Cleaning & Maintenance

1. Warning : To avoid risk of short-circuit or electric shock, do not disassemble the Adapter and the breast pump or immerse it into water. Unplug the Adapter before cleaning.
2. Clean the surface of the breast pump with clean damp cloth.
3. Do not use detergent or corrosive agents to clean the breast pump and the Adapter.

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